



| TEST  | DESCRIPTION  | PASS RESPONSE   | PASS/FAIL   |
|---|--|---|---|
| <b>Assessment Level 1</b><br>Assessment of: <ul style="list-style-type: none"> <li>Sitting balance</li> <li>Upper extremity and core strength</li> <li>Orthostatic response to sitting</li> <li>Ability to follow directions</li> </ul> | <b>Assessment Level 1 – Sit and Shake – verifies hemodynamic stability and sitting balance.</b><br><br><b>Sit and Shake:</b> From semi-reclined or sitting position, ask patient to sit upright for up to 1 minute (as indicated based on concern for orthostatic hypotension or postural intolerance); then reach across midline and shake hands with caregiver – repeat with other hand. (Patient’s feet may or may not be flat on the floor)<br><br><b>Safe Mode:</b> Use sling and lift to assist to side of bed or complete in bed with HOB raised and patient leaning forward or with bed in chair mode. | <b>Sit and Shake:</b> <ul style="list-style-type: none"> <li>Able to follow commands and sit unsupported (by sling or surface) for up to 1 minute.</li> <li>Able to maintain seated balance while reaching across midline of trunk with one or both hands and shaking caregiver’s hand.</li> </ul> <p><i>NOTE: If patient has "strict bed rest" orders, or is hemodynamically unstable, patient is a Mobility Level 1. Do not proceed with the assessment.</i></p>  | <b>FAIL = Mobility Level 1</b><br>Goal: Avoid complications of immobility <ol style="list-style-type: none"> <li>EOB dangling with sling and lift; calf pump exercises</li> <li>Bed in Fowlers or chair position</li> <li>Lift and repo sheet – boosting and turning</li> <li>Lift and Multistraps – turning and limb holding</li> <li>Lift and sling – bed to chair/ commode transfers</li> <li>Friction Reducing Device (FRD) for AROM/PROM</li> </ol> <p><b>PASS</b> – Proceed to Assessment Level 2</p>   |
| <b>Assessment Level 2</b><br>Assessment of: <ul style="list-style-type: none"> <li>Lower extremity strength in preparation for weight bearing</li> <li>Foot drop</li> </ul>   | <b>Assessment Level 2 – Stretch – verifies some lower extremity strength.</b><br><br><b>Patient demonstrates Level 1 function and lower extremity strength.</b><br><b>Stretch:</b> While sitting upright unsupported, extend leg by straightening knee, then point toes/ pump ankle between dorsiflexion/plantar flexion 3 times. Knee remains below hip level.<br><br><b>Safe Mode:</b> Use seated sling with lift (mobile or overhead) or bed in chair position.   | <b>Stretch:</b><br>Able to extend leg and straighten knee to engage quadriceps; then able to pump ankle for 3 repetitions to engage lower leg muscles and assist with venous return/fluid shift.   | <b>FAIL = Mobility Level 2</b><br>Goal: Avoid Complications of Immobility <ol style="list-style-type: none"> <li>FRD: partial squats and leg AROM exercises</li> <li>Lift and repo sheet – boosting and turning</li> <li>Lift and Multistraps™ – turning and limb holding</li> <li>Lift and sling – bed to chair/ commode transfers</li> <li>Ankle pumps</li> </ol> <p><b>PASS</b> – Proceed to Assessment Level 3</p>  |
| <b>Assessment Level 3</b><br>Assessment of: <ul style="list-style-type: none"> <li>Ability to perform sit to stand transfer</li> <li>Static standing balance</li> <li>Orthostatic response to standing</li> </ul>                       | <b>Assessment Level 3 – Stand – verifies patient has adequate strength, physiological stability and balance to stand.</b><br><br><b>Stand:</b> With feet flat on the floor about shoulder width apart, ask patient to shift weight forward and come to standing position. Stand for a minimum of 5 seconds and up to 1 minute if there is concern for orthostatic hypertension or syncope.<br><br><b>Safe Mode:</b> Use sit-to-stand lift or ambulation vest/pants with lift.  | <b>Stand:</b><br>Able to rise, maintain standing position for up to 1 minute.<br><br><p><i>NOTE: The majority of patients who exhibit orthostatic hypotension do so within the first minute of standing.</i></p> May use walker, cane, crutches or prosthetic leg(s) as appropriate.   | <b>FAIL = Mobility Level 3</b><br>Goal: Strengthen, assist fluid shifts, avoid falls <ol style="list-style-type: none"> <li>Sit-to-stand lift – stand for 1-2 minutes, weight shift side to side and forward/back, 2-3 deep breaths</li> <li>Squats using FRD with bed in tilt position</li> <li>Lifts and Multistraps™: limb holding</li> <li>Sit-to-stand lift (powered or non-powered) chair/ commode transfers</li> <li>Progress to standing with sit-to-stand lift to standing with aid.</li> </ol> <p><b>PASS</b> – Proceed to Assessment Level 4</p>       |
| <b>Assessment Level 4</b><br>Assessment of: <ul style="list-style-type: none"> <li>Pre-ambulation weight shifting</li> <li>Dynamic standing balance</li> <li>Motor planning and ability to follow directions</li> </ul>                 | <b>Assessment Level 4 – Step – verifies adequate leg strength, balance and control to lift foot off floor and advance forward/back prior to walking.</b><br><br><b>Step:</b> <ol style="list-style-type: none"> <li>March or step in place (not high marching)</li> <li>Step forward with one foot, weight bear/ shift weight onto foot and return to starting position; repeat with other foot.</li> </ol> <b>Safe Mode:</b> Use ambulation vest/pants and lift.<br><br>Always default to Safe Mode if concerned regarding orthostatic hypotension/syncope or falls.  | <b>Step:</b><br>Able to perform both marching-in-place and step forward/back with each leg.<br><br>May use walker, cane, crutches, or prosthetic leg(s) as appropriate.    | <b>FAIL = Mobility Level 4</b><br>Goal: Improve standing tolerance, balance; avoid falls; consider mobility goals <ol style="list-style-type: none"> <li>Ambulation vest/pants and lift for standing, pre-gait and ambulation activities</li> <li>Set ambulation distance and frequency goals after passing 'Step'</li> <li>If using aid to pass 'Step,' assure that aid is always accessible and used</li> </ol> <p><b>PASS</b> – Pass BMAT Assessment; Proceed through Discharge Planning; Continue to complete once per shift and with change in condition</p> |

Adapted from Boynton, et al., T. (2020). The Bedside Mobility Assessment Tool 2.0. American Nurse Journal, 15.

Hillrom offers assessment training aligned with BMAT principles and clinical application. Contact your local representative for information.

[hillrom.com](http://hillrom.com)

Hill-Rom reserves the right to make changes without notice in design, specifications and models. The only warranty Hill-Rom makes is the express written warranty extended on the sale or rental of its products.